



2K full-color clear imaging, reducing the rate of manual microscopic review for positive samples

Colorful imaging technique which is close to manual microscopic satisfies what you need for releasing a report

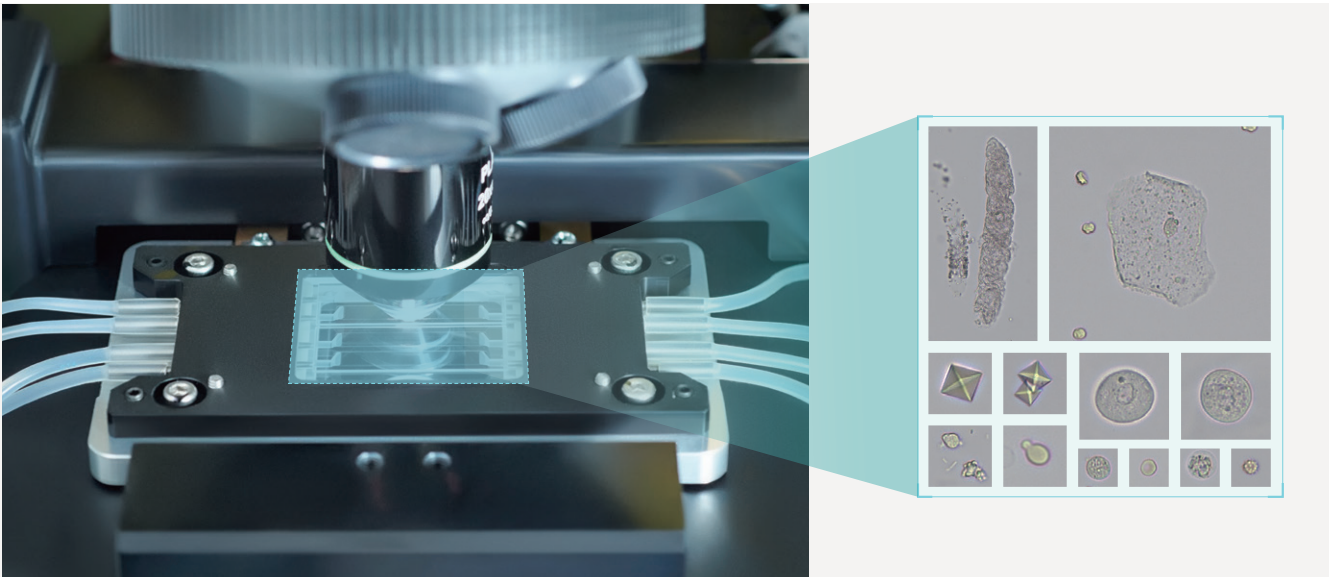

High analysis volume for sediment particles to avoid missed diagnosis of positive samples

The system captures up to 60 clear images from the samples. Utilizing advanced algorithm, the system can precisely identify various types of formed elements from panoramic images, thereby minimizing the chances of missed positive samples.

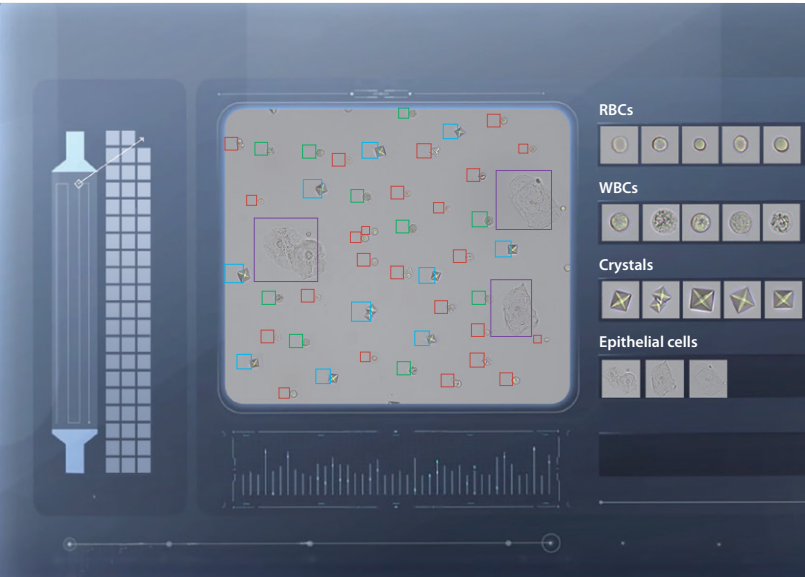

EU-5300 Pro
Automated Urinalysis System

Testing principles

Dry chemistry: Photoelectric colorimetry, refractometry, and Nephelometry method.
Formed elements: Machine vision (Digital imaging technology)

Reportable parameters

Physical: Specific gravity, color, and turbidity
Dry chemistry: 11/14 items
LEU, leukocyte NIT, nitrite
URO, urobilinogen pH, power of hydrogen
PRO, protein BLD, erythrocyte
BIL, bilirubin MCA, micro albumin
GLU, glucose CRE, creatinine
VitC, vitamin C CA, calcium
SG, specific gravity P/C, protein-to-creatinine ratio*
KET, ketones A/C, microalbumin-to-creatinine ratio*
Note: Parameters indicated with an asterisk (*) are intended for calculated parameters

Formed elements: 31 kinds
RBCs (total) BACT, bacteria
Normocytic, Nor-RBC BACTc, coccus
Macrocyte, Mac-RBC BACTr, rod
Microcyte, Mic-RBC YST, Yeast
Crenocyte, Cre-RBC Squamous epithelial cell, SEC
Annular RBCs, Ann-RBC Non squamous epithelial cell, NEC
Acanthocytes, Aca-RBC CRYs, Crystals
Humped Spherocytes, Hpd-RBC Mono-hydrate calcium oxalate crystal, Caoxm
Jagged RBCs, Jag-RBC Di-hydrate calcium oxalate crystal, Caoxd
Ghost RBCs, Gho-RBC Uratic crystal, URA
Fragmented RBC, Fra-RBC Triple phosphate crystal TRP
Other abnormal RBCs, Oab-RBC HYAC, hyaline cast
Anisocytosis Ratio, Ani-Ratio UNCC, unclassified cast
MorInfo-RBC MUC, mucous strands
WBC, white blood cells Sperm, SPRM
WBCC, white blood cells clump

Throughput

Dry chemistry mode: ≥160 tests/ hour
Formed element mode: ≥70 tests/ hour
Hybrid mode: ≥70 tests/ hour

Automated focusing

The system automatically performs the focusing process, without the need for manual operation or the use of focus reagents

STAT function

Independent STAT positions for emergency samples at any time

Strips loading capacity

200

Sample Volume

2.5ml minimum required sample volume

Sample type

Native urine sample

Barcode

Built-in barcode reader

Tube specification

Length ≤110mm Diameter 15-16mm

Communication

Bi-direction LIS, automated barcode identification for testing

Dimensions and Weight

WxDxH: 600x770x530(mm) / ≤71.5Kg



EU-5300 Pro
Automated Urinalysis System

Unveiling the Unseen by
Unlocking Unparalleled Clarity





Three features integrated in a single system to manage the unique workflow needs of your lab

The integration of dry chemistry, formed element, and RBC phase analysis leads to a great improvement in time and space management efficiency. Even a single person can easily manage the TAT requirements during the peak hours of your lab.




Coming standard with RBC phase parameters to help rapidly identify the source of hematuria

The system can automatically provide parameters for various types of urinary RBCs, presenting histograms that clearly depict the size, shape, and hemoglobin content of these cells. By analyzing the results of RBC morphological variations (MorInfo-RBC: homogeneous/heterogeneous/mixed) and utilizing RBC morphological images, a thorough review and validation process can be conducted to identify the source of hematuria rapidly and precisely.

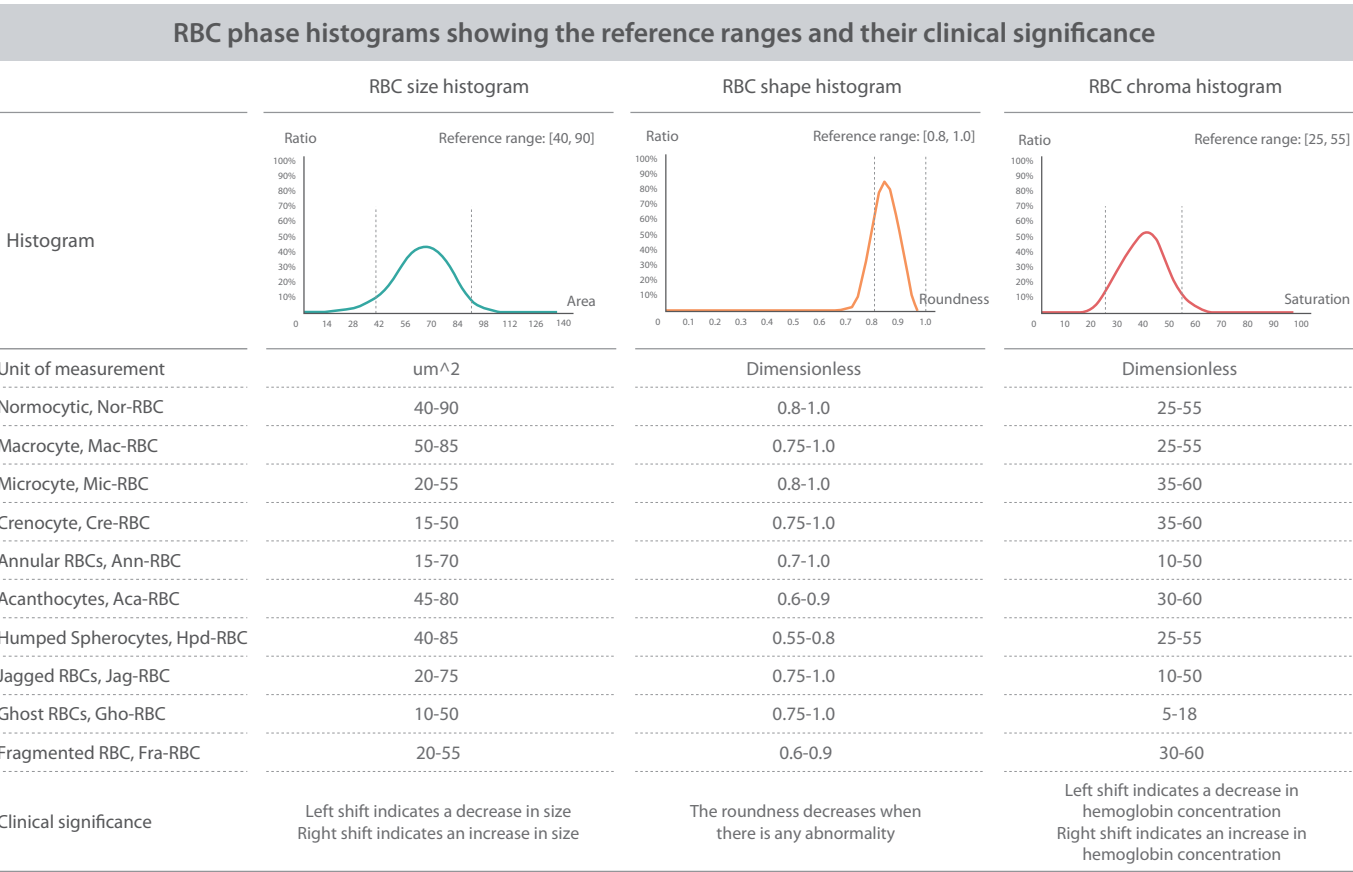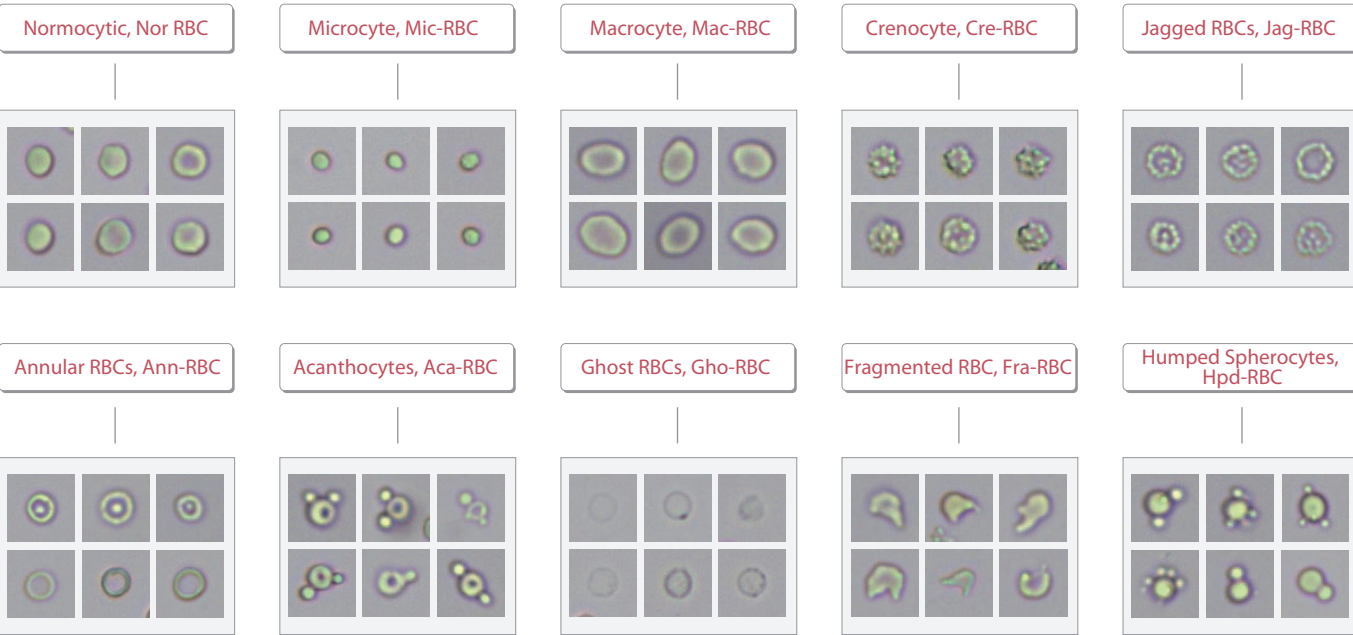

Dry chemistry analysis delivers precise results that are better aligned with the analysis of formed elements

The test strip for dry chemistry analysis are optimized to enable precise sampling, thereby ensuring accurate results, consistent performance, and greater ease of use.

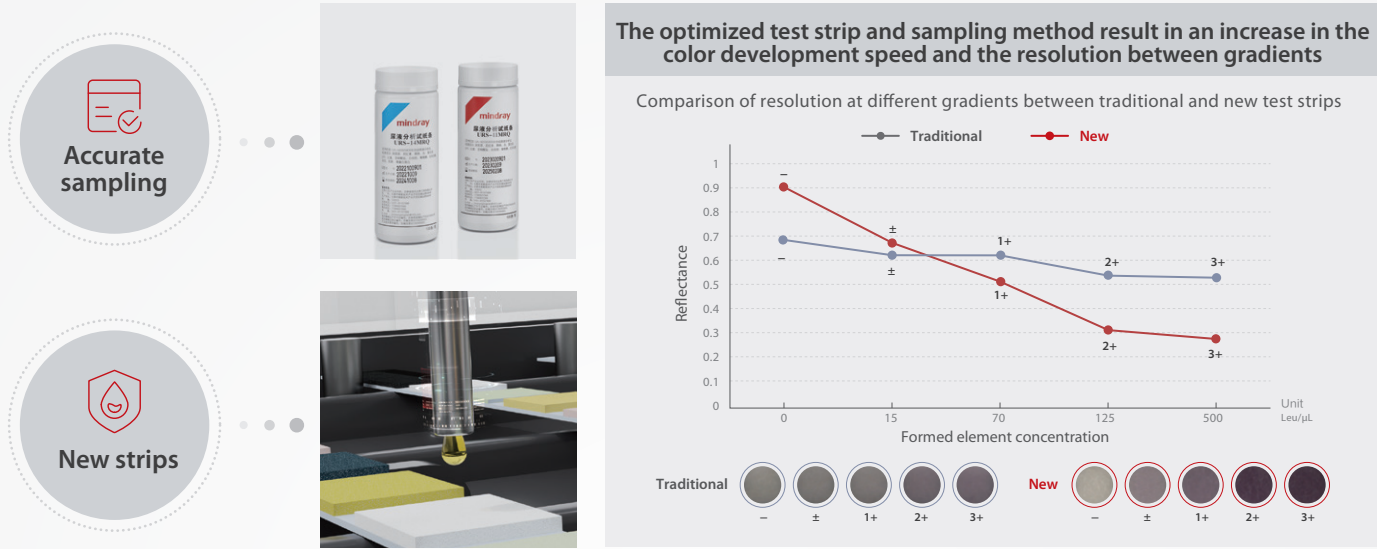

Safety and ease of use, and efficient review

The system supports running urine samples with closed tubes, ensuring enhanced levels of biosafety. With the inclusion of Mindray data management software—LabXpert, the system shows dry chemistry, urine formed elements, and RBC phase results on a single screen, significantly improving the efficiency of report validation.

